TEOSYAL 30G Global Action

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Description

TEOSYAL 30G Global Action is a transparent, non pyrogenic, visco-elastic gel of cross-linked hyaluronic acid, of non animal origin. Each box contains one or two pre-filled syringes of TEOSYAL 30G Global Action.

Composition Cross-linked hyaluronic acid 25 mg Phosphate buffer pH 7.3 qsp 1 ml

Indications The therapeutic purposes of TEOSYAL products are: - to modify the anatomy of aged skin: restoration of volumes, filling of skin depression and wrinkles, restoration of skin hydration, - reconstructive surgery: filling of depressions due to scars, reconstruction of volumes lost by lipatrophy.

Mode of action TEOSYAL 30G Global Action is injected into the mid dermis at the site of the cutaneous depression to be corrected. TEOSYAL 30G Global Action creates a volume that corrects wrinkles.

Contra-indications - for the correction of superficial wrinkles - for injections other than intradermal

Posology and method of administration The injection of TEOSYAL 30G Global Action is reserved for practitioners trained in the injection techniques of products intended for the filling of wrinkles.

Side effects The practitioner must inform the patient that there are potential side effects related to the implantation of this device, occurring immediately following injection or after a delay.

Assembly of the needle on the syringe For optimum use of TEOSYAL products, it is important for the needle to be assembled on the syringe following the 4 steps detailed in the diagrams (see figures 1 to 5).

Precautions for use Verify the expiry date and the integrity of the packaging before use. Do not use if the expiry date is exceeded or if the packaging is damaged.

Storage conditions Store between 2°C and 25°C, away from direct sunlight. Make sure there are no visible signs of damage to the packaging before use.

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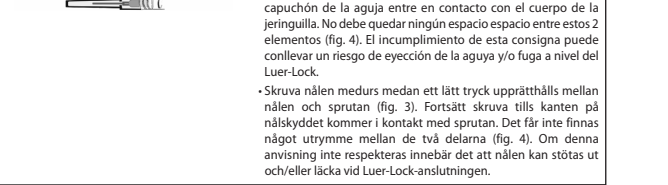
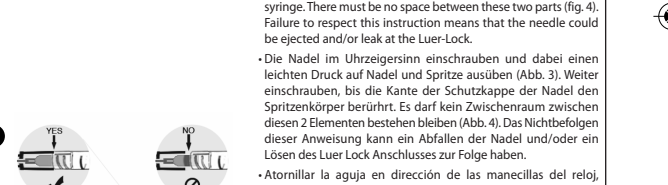
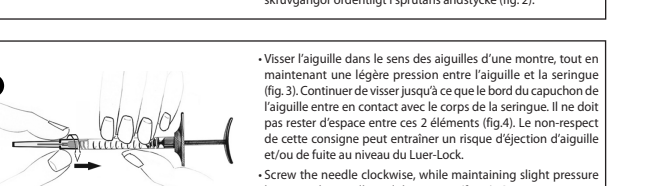
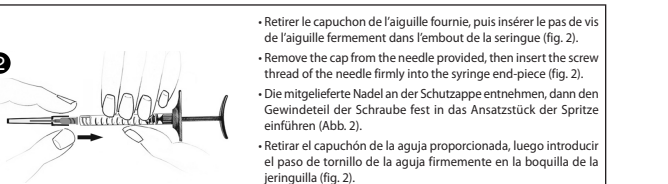
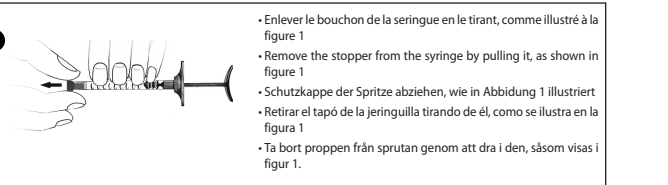
Remove the stopper from the syringe by pulling it, as shown in figure 1. Remove the cap from the needle provided, then insert the screw thread of the needle firmly into the syringe end-piece. Continue screwing until the edge of the cap of the needle contacts the body of the syringe. Tighten the needle clockwise, while maintaining slight pressure between the needle and the syringe. Firmly holding the body of the syringe with the other hand, remove the needle protection by pulling it firmly with the hand while holding the body of the syringe.