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GB Instruction leaflet

Description

Teosyal MesoExpert is a viscoelastic gel of non cross-linked hyaluronic acid - sterile, non pyrogenic, colourless and of non-animal origin. Each box contains one pre-filled syringe of Teosyal MesoExpert. For each syringe, the box contains two sterile 30G1/2" needles and two tracability labels (one to be given to the patient, and one to be kept by the doctor in the patient's file). The volume of each syringe is shown on the packaging carton as well as on each syringe.

Composition:

Hyaluronic acid 15 mg
Supplemented* phosphate buffer pH 7,3 qsp 1 ml
*the specific excipients present in the supplemented phosphate buffer are alpha-lipoic acid, glutathione, N-acetyl-L-cystéine, L-arginine, L-isoleucine, L-leucine, L-lysine monohydrate, glycine, L-valine, L-threonine, L-proline, pyridoxine hydrochloride, zinc acetate dihydrate and copper sulfate pentahydrate.

Indications

The therapeutic purposes of Teosyal® products are:
- to modify the anatomy of aged skin: restoration of volumes, filling of skin depression and wrinkle, restoration of skin hydration,
- reconstructive surgery: filling of depressions due to scars, reconstruction of volumes lost by liposuction.
Teosyal MesoExpert is indicated for the prevention of wrinkles and rehydration of the face, neck and neckline.

Mode of action

Hyaluronic acid is a major constituent of ground substance. A veritable molecular sponge, its helicoïdal structure enables it to trap one thousand times its weight in water. It is widely distributed in cutaneous tissue and hydrates the skin, though its content diminishes with age. As the skin thins and becomes fragile, the initial skin fractures appear. Teosyal MesoExpert is a hyaluronic acid gel which is injected into the dermis. Thanks to its marked fluidity and its visco-elastic properties, Teosyal MesoExpert diffuses uniformly into cutaneous tissue, maintaining its hydration and reinforcing the dermis. Teosyal MesoExpert is biodegradable and is absorbed over time. Periodic treatment sessions are necessary to obtain a satisfactory level of skin hydration.

Contra-indications

Teosyal MesoExpert must not be used:
- for injections other than intradermal
- in combination with peeling, laser treatment or ultrasound-based,
- if the patient has cutaneous disorders, inflammation or infection at the treatment site or near to this site.
- in the case of patients having a known hypersensitivity to hyaluronic acid, with a history of severe allergic or anaphylactic shock
- in case of patients with autoimmune diseases,
- in pregnancy, breast-feeding mothers, or in children
Do not inject into blood vessels

Posology and method of administration

The injection of Teosyal MesoExpert is reserved for practitioners trained in the injection techniques of filling products. Before beginning treatment, patients must be questioned regarding their past medical history, informed regarding the foreseeable outcome of treatment and of potential undesirable effects. Treatment sites must be properly disinfected. It is advised to perform injections using the needles provided in the box. Teosyal MesoExpert should be injected slowly and superficially over the treatment sites using the multipuncture technique with the aid of the 30G1/2" needles provided. Manually stretch the treatment sites and carry out multipunctures of 2 to three millimetres in diameter separated by one centimetre to a depth of approximately two millimetres. The volume to be injected depends on the size of the area to be treated. It is generally advised to count 2 to 3 treatment sessions at an interval of 3 or 4 weeks, followed by a session every 2 to 3 months in order to maintain the level of hydration.

Side effects:

The practitioner must inform the patient that there are potential side effects related to the implantation of this device, occurring immediately following injection or after a delay. These include, among others:
- Local manifestations: inflammatory reactions (erythema, oedema, pain at the point of injection), hematomas, itching, temporary loss of sensitivity around the injected area, dyschromia, abscesses, indurations, nodules (possibly granulomas), skin necrosis, migration of the implant.
- General manifestations: immediate hypersensitivity up to anaphylactic shock.
The patient should inform the practitioner of any secondary effect other than those described above or any effect persisting beyond one week.
The practitioner will, in turn, inform the product retailer within the shortest time possible.

Assembly the needle on the syringe

For optimum use of Teosyal® products, it is important for the needle to be assembly on the syringe following the 4 steps detailed in the diagrams (see figures 1 to 5). Stop the injection and change the needle if you feel an obstruction or pressure during the injection.

Precautions for use

Verify the expiry date and the integrity of the packaging before use. Do not use if the expiry date is exceeded or if the packaging is damaged. You should warn the patient against taking high-dose Vitamin E, aspirin, anti-inflammatory or anticoagulants, during the week before the injections session. Advise the patient not to use make-up during the 12 hours following injection and to avoid extreme temperatures (intense cold, sauna,...) during the week following treatment. At the end of the treatment session, it is essential to discard all remaining unused product. This device is intended for single use only. Sterility is not guaranteed in the event of re-use. The gel can deteriorate, reducing its efficiency and the gel can dry out, hindering its extrusion through a needle.
Dispose of used needles in appropriate containers. Dispose of used needles in appropriate containers.

Storage conditions

Store between 2°C and 25°C, away from direct sunlight.
Make sure there are no visible signs of damage to the packaging before use.

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