TEOSYAL® RHA 2

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GB Instruction leaflet

Description
TEOSYAL® RHA 2 is a sterile, transparent, non pyrogenic, visco-elastic gel of cross-linked hyaluronic acid of non animal origin containing 0.3% lidocaine hydrochloride by mass for its anaesthetic properties. Each box contains two syringes prefilled with TEOSYAL® RHA 2. For each syringe, the box contains one or two sterile 30G1/2" needles and two traceability labels (one to be given to the patient, and one to be kept by the doctor in the patient's file). The volume of each syringe is shown on the box as well as on each syringe.

Composition
Cross-linked hyaluronic acid 23 mg
Lidocaine hydrochloride 3 mg
Phosphate buffer pH 7.3 qs ad 1 ml

Indications
The therapeutic purposes of TEOSYAL® products are:
- to modify the anatomy of aged skin: restoration of volumes, filling of skin depression and wrinkle, restoration of skin hydration, reconstructive surgery: filling of depressions due to scars, reconstruction of volumes lost by lipotrophy. TEOSYAL® RHA 2 is indicated to fill wrinkles on the face, damaged skin such as mild or moderate nasogenian folds, peribuccal and glabella wrinkles. Lidocaine is a local anaesthetic designed to reduce the sensation of pain.

TEOSYAL® RHA 2 creates a volume that corrects wrinkles.
TEOSYAL® RHA 2 is biodegradable and is slowly resorbed over time. In general one, but sometimes two treatment sessions are necessary, depending upon the depth of the wrinkles in need of treatment, to obtain an optimal degree of correction. Periodic touch-up sessions then enable the sought-after degree of correction to be maintained. Sensitivity during the injection may be different and vary from person to person depending on the area to be treated and the injection technique used.

Mode of action
TEOSYAL® RHA 2 is injected into the mid dermis at the site of the cutaneous depression in need of correction. TEOSYAL® RHA 2 thus creates a volume which fills the wrinkles. TEOSYAL® RHA 2 is biodegradable and is slowly resorbed over time. In general one, but sometimes two treatment sessions are necessary, depending upon the depth of the wrinkles in need of treatment, to obtain an optimal degree of correction. Periodic touch-up sessions then enable the sought-after degree of correction to be maintained. Sensitivity during the injection may be different and vary from person to person depending on the area to be treated and the injection technique used.

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Contra-indications
TEOSYAL® RHA 2 must not be used:
- for the correction of superficial wrinkles
- for injections other than intradermal
- for injections in the eyelids
- if the patient has cutaneous disorders, inflammation or infection at the treatment site or near to this site
- in the case of patients having a known hypersensitivity to hyaluronic acid, with a history of severe allergy or anaphylactic shock
- in the case of known hypersensitivity to lidocaine and/or amide local anaesthetic agents
- in combination with peeling, laser or ultrasound-based treatments
- in the case of patients with hepato-renal insufficiency and/or undergoing treatment for liver disease (beta blockers)
- for patients suffering from epilepsy or porphyria due to possible interactions with other filling implants, which have not been researched, it is inadvisable to inject TEOSYAL® RHA 2 into sites in the presence of other filling implants
- in pregnancy, breast-feeding mothers, or in children
Sportsmen and women must be made aware of the fact that this product contains an active component which may cause a positive result in drug tests.
Do not inject into blood vessels.

Caution
The practitioner must inform the patient that there are potential side effects related to the implantation of this device, occurring immediately following injection or after a delay. These include, among others:
- Local manifestations: inflammatory reactions (erythema, oedema, pain at the point of injection), hematomas, itching, temporary loss of sensitivity around the injected area, dyschromia, abscesses, indurations, nodules (possibly granulomas), skin necrosis, migration of the implant.
- General manifestations: immediate hypersensitivity up to anaphylactic shock.
The patient should inform the practitioner of any secondary effect other than those described above or any effect persisting beyond the expected duration of action. This device is intended for single use only. Sterility is not guaranteed in the event of re-use. The gel can deteriorate, reducing its efficiency and the gel can dry out, hindering its extrusion through a needle.
Dispose of used needles in appropriate containers.
Store between 2°C and 25°C, away from direct sunlight. Make sure there are no visible signs of damage to the packaging before use.

Usage and method of administration
The injection of TEOSYAL® RHA 2 is reserved for practitioners trained in the injection techniques of products intended for the filling of wrinkles. Before beginning treatment, patients must be questioned regarding their past medical history and informed regarding the foreseeable outcome of treatment and of potential undesirable effects. Correction sites must be properly disinfected. It is advised to perform injections using the needle provided in the box. TEOSYAL® RHA 2 should be injected slowly into the mid dermis using the linear threading injection technique. If the injection is made too deeply, i.e. into subcutaneous tissue, the correction will not have the sought-after duration of action. An injection which is too deep will encounter subcutaneous tissue which will not give rise to resistance during injection of the product, unlike the dermis. If the colour of the needle is visible under the skin during injection, this indicates that the injection will be too superficial, which it is also advisable to avoid, as the result of the correction might be irregular. The injection volume depends upon the correction required. It is however recommended not to exceed 3 ml per session. It is important not to overcorrect. Massage the treated sites carefully in order to ensure a uniform distribution of the product at the corrected sites.

Side effects
The practitioner must inform the patient that there are potential side effects related to the implantation of this device, occurring immediately following injection or after a delay. These include, among others:
- Local manifestations: inflammatory reactions (erythema, oedema, pain at the point of injection), hematomas, itching, temporary loss of sensitivity around the injected area, dyschromia, abscesses, indurations, nodules (possibly granulomas), skin necrosis, migration of the implant.
- General manifestations: immediate hypersensitivity up to anaphylactic shock.
The patient should inform the practitioner of any secondary effect other than those described above or any effect persisting beyond the expected duration of action. This device is intended for single use only. Sterility is not guaranteed in the event of re-use. The gel can deteriorate, reducing its efficiency and the gel can dry out, hindering its extrusion through a needle.
Dispose of used needles in appropriate containers.
Store between 2°C and 25°C, away from direct sunlight. Make sure there are no visible signs of damage to the packaging before use.

Mounting the needle on the syringe
For optimum use of TEOSYAL® RHA products, it is important for the needle to be mounted on the syringe following the 4 steps detailed in the diagrams (See figures 1 to 5).

Precautions for use
The practitioner must take into account the fact that this filling product contains lidocaine. Check the use-by date and the integrity of the packaging before use. Do not use after the use-by date or if the packaging is damaged. The patient should be advised not to take high-dose vitamin E, aspirin, anti-inflammatory or anti-coagulants during the week before the injection session. Advise the patient not to use make-up during the 12 hours following injection and to avoid extreme temperatures (intense cold, sauna, hammam, etc.) during the week following injection. At the end of the treatment session, it is essential to discard all remaining unused product. This device is intended for single use only. Sterility is not guaranteed in the event of re-use. The gel can deteriorate, reducing its efficiency and the gel can dry out, hindering its extrusion through a needle.
Dispose of used needles in appropriate containers.
Store between 2°C and 25°C, away from direct sunlight. Make sure there are no visible signs of damage to the packaging before use.

Storage conditions
Store between 2°C and 25°C, away from direct sunlight. Make sure there are no visible signs of damage to the packaging before use.

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