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Instruction leaflet

Description

Teosyal Ultimate is a transparent, non pyrogenic, visco-elastic gel of cross-linked hyaluronic acid, of non animal origin. Each box contains one syringe pre-filled with Teosyal Ultimate, two sterile 23G1* needles and two traceability labels (one to be given to the patient, one to be kept by the doctor in the patient's file). The volume of each syringe is shown on the box as well as on each syringe.

Composition

Acide hyaluronique réticulé 22 mg
Phosphate buffer pH 7,3 q.s. ad 1 ml

Indications

The therapeutic purposes of Teosyal® products are: modification of the anatomy of aged skin: restoration of volumes, filling of skin depression and wrinkle, restoration of skin hydration, - reconstructive surgery: filling of depressions due to scars, reconstruction of volumes lost by lipotrophy. Teosyal Ultimate is indicated for adding volume to facial tissue, in areas such as the cheeks, chin and other areas of volume loss. Facial contours are remodelled.

Mode of action

Teosyal Ultimate is injected at variable depth depending on the area to be treated: this varies from fatty subcutaneous tissue to pre-riosteal facial areas. Teosyal Ultimate creates a volume that fills the cutaneous depression and restores the contours of the face. Teosyal Ultimate is biodegradable and is slowly resorbed over time. Generally, one or sometimes two treatment sessions are necessary, depending upon the depth of the injection and the area to be treated, to obtain an optimal degree of correction. The effectiveness of the correction depends on many parameters such as the injection technique, as well as the nature and elasticity of the area to be treated.

Contra-indications

Teosyal Ultimate must not be used: - for injections other than subcutaneous and/or pre-riosteal injections - in combination with peeling, laser or ultrasound-based treatments - if the patient has cutaneous disorders, inflammation or infection at the treatment site or near to this site. - in the case of patients having a known hypersensitivity to hyaluronic acid, with a history of severe allergy or anaphylactic shock. - in case of patients with autoimmune diseases - since interactions with other filling implants have not been researched, it is inadvisable to inject Teosyal Ultimate into sites with other implants of facial tissue. - in pregnancy, breast-feeding mothers, or in children - Do not inject into blood vessels

Dosage and method of administration

The injection of Teosyal Ultimate is reserved for practitioners trained in sub-cutaneous and/or pre-riosteal injection techniques for adding volume to the sub-cutaneous facial tissue. Before beginning treatment, patients will be required to complete a medical history and must be informed of the fore-mentioned risks associated with the injection. The doctor should check the possibility of using either local or regional anaesthetic block. The areas to be treated must be disinfected with a suitable antiseptic solution. Teosyal Ultimate should be injected using hypodermic needles of size 23G1*, such as the needles supplied with the product. Depending on the injection technique, it is also possible to use a round cannula with side holes, available from the distributor.

Inject the product slowly ensuring the needle advances along the desired trajectory, checking the position of the needle with your free hand and adjusting the quantity of product injected. The injection technique may vary according to the area to be treated and the quantity administered. It is therefore essential that the practitioner be qualified in the techniques of sub-cutaneous volume augmentation. Teosyal Ultimate should be injected using hypodermic needles of size 23G1*, such as the needles supplied with the product. Depending on the injection technique, it is also possible to use a round cannula with side holes, available from the distributor.

Inject the product slowly ensuring the needle advances along the desired trajectory, checking the position of the needle with your free hand and adjusting the quantity of product injected. The injection technique may vary according to the area to be treated and the quantity administered. It is therefore essential that the practitioner be qualified in the techniques of sub-cutaneous volume augmentation. Teosyal Ultimate should be injected using hypodermic needles of size 23G1*, such as the needles supplied with the product. Depending on the injection technique, it is also possible to use a round cannula with side holes, available from the distributor.

Side effects

The practitioner must inform the patient that there are potential side effects related to the implantation of this device, occurring immediately following injection or after a delay. These include, among others: - Local manifestations: inflammatory reactions (erythema, oedema, pain at the point of injection), haematomas, itching, loss of sensitivity around the injected area, dyschromia, abscesses, indurations, nodules (possibly granulomas), skin necrosis, migration of the implant. - General manifestations: immediate hypersensitivity up to anaphylactic shock. The patient should inform the practitioner of any secondary effect other than those described above or any effect persisting beyond one session.

The practitioner will, in turn, inform the product retailer within the shortest time possible.

Assembly the needle on the syringe

For optimum use of Teosyal® products, it is important for the needle to be assembled on the syringe following the 4 steps detailed in the diagrams (see figures 1 to 5). Stop the injection and change the needle if you feel an obstruction or pressure during the injection.

Precautions for use

Verify the expiry date and the integrity of the packaging before use. Do not use if the expiry date is exceeded or if the packaging is damaged. You should warn the patient not to take high-dose Vitamin E, aspirin, anti-inflammatory or anticoagulants the week before the injections session. Advise the patient not to use make-up during the 12 hours following injection and to avoid extreme temperatures (intense cold, sauna, etc.) during the week following treatment. At the end of the treatment session, it is essential to discard all remaining unused product. This device is intended for single use only. Sterility is not guaranteed in the event of reuse. The gel can deteriorate, reducing its efficiency and the gel can dry out, hindering its extrusion through a needle. The implantation of Teosyal Ultimate consists of a subcutaneous and/or pre-riosteal injection. It is therefore a medical procedure which involves an inherent risk of infection, like any procedure of this type. It must be performed in a suitable environment and all the usual precautions must be taken. In addition, the medical practitioner must have good knowledge of the facial anatomy. Dispose of used needles in appropriate containers.

Storage conditions

Store between 2°C and 25°C, away from direct sunlight. Make sure there are no visible signs of damage to the packaging before use.

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